



FDA Cleared

HSA Approval: Pending

## Multiple Drug Rapid Cup Test

5 mins

One-step simultaneous detection of 17 drugs of abuse in urine.

### — FEATURES & BENEFITS

- Just one-step, no training required
- Result in 5 minutes
- Simultaneous detection of 17 drugs
- Built-in tamper detection to ensure test integrity
- Room temperature storage, no refrigeration

### — 17-PANEL DRUG SCREEN

| CODE | SUBSTANCE            | CUT-OFF<br>(ng/mL) | CODE  | SUBSTANCE                 | CUT-OFF<br>(ng/mL) |
|------|----------------------|--------------------|-------|---------------------------|--------------------|
| AMP  | Amphetamine          | 500/1000           | MTD   | Methadone                 | 300                |
| BAR  | Barbiturates         | 300                | OXY   | Oxycodone                 | 100                |
| BUP  | Buprenorphine        | 10                 | PCP   | Phencyclidine             | 25                 |
| BZO  | Benzodiazepines      | 300                | PPX   | Propoxyphene              | 300                |
| COC  | Cocaine              | 150/300            | TCA   | Tricyclic Antidepressants | 1000               |
| EDDP | Methadone Metabolite | 300                | THC   | Marijuana                 | 50                 |
| FYL  | Fentanyl             | 10                 | 6-MAM | Heroin Metabolite         | 10                 |
| MET  | Methamphetamine      | 500/1000           |       |                           |                    |
| MDMA | Ecstasy              | 500                |       |                           |                    |
| MOP  | Morphine / Opiates   | 300/2000           |       |                           |                    |

## – OPERATING PROCEDURE

### STEP 1



Remove the test cup.

### STEP 2



Urinate directly into  
the test cup.

### STEP 3



Close the lid tightly.

### STEP 4



2 MINUTES

Read the urine  
adulteration  
test results.

| TEST                       | ABNORMAL<br>(LOW) | NORMAL                  | ABNORMAL<br>(HIGH) |
|----------------------------|-------------------|-------------------------|--------------------|
| Oxidant/<br>PCC            |                   | Recycled                |                    |
| Specific<br>Gravity (S.G.) | 1.000             | 1.005 1.010 1.015 1.020 | 1.030              |
| pH                         | 2.0 3.0           | 4.0 4.0 7.0             | 10 11 12           |
| Nitrite<br>(mg/dL)         |                   | 10 20                   | 10 100             |
| Glutaraldehyde<br>(mg/dL)  |                   |                         |                    |
| Creatinine<br>(mg/dL)      | 0                 | 20 100                  |                    |

### STEP 5



Peel off the label  
to reveal the  
test result.

### STEP 6




5 MINUTES

Read the test  
result at  
5 minutes.



#### POSITIVE

Two lines appear.  
One at C and one at T,  
no matter how faint  
the T line is.

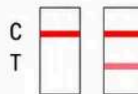

#### NEGATIVE

Only one line appears at C.  
No line appears at T.



#### INVALID

No line appears at C,  
regardless of whether a  
line appears at T.

**Note:** Do not read the result after 5 minutes.